



Innovation in Spirometry Oximetry Telemedicine

Registration and installation Guide MIR Spiro software

www.spirometry.com
www.mirsmartone.com

Thank you for choosing a MIR Medical International Research product.

If there is a problem with this software, please contact your distributor or the manufacturer.

The manufacturer continuously develops and improves its products. The information contained in this manual is therefore subject to change without notice. The manufacturer cannot be held responsible for any loss or damage caused by misuse of the product or this manual.

As printing has its limitations, the images or symbols contained may differ from those displayed on the device.


Any copy, partial or complete, of this manual is strictly prohibited.

Registration or connection to MyMir

In order to be able to download and install the MIR Spiro software you must create your MyMir account on the site www.spirometry.com by selecting the icon  then MyMir or by connecting to www.mymir.spirometry.com


The page below is displayed. If you are already registered on our site, you only need to fill in the login information.

If you have lost your password, press “**Forgot your password?**”

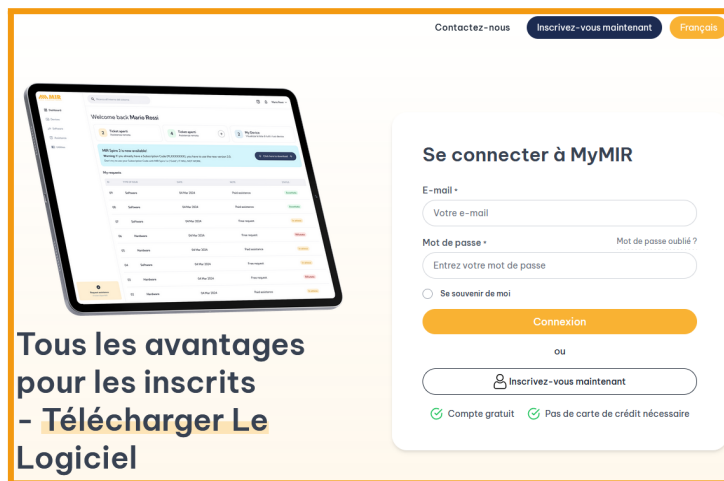

The screenshot shows the MyMir login interface. At the top, there are links for 'Contactez-nous', 'Inscrivez-vous maintenant', and 'Français'. On the left, a tablet displays the 'Tous les avantages pour les inscrits' section, which includes a 'Télécharger Le Logiciel' button. The main area is titled 'Se connecter à MyMIR' and contains a login form with fields for 'E-mail' and 'Mot de passe'. There is a 'Mot de passe oublié ?' link next to the password field. Below the fields is a 'Connexion' button. Underneath the button is a link to 'Inscrivez-vous maintenant'. At the bottom, there are two green checkmarks indicating 'Compte gratuit' and 'Pas de carte de crédit nécessaire'.

If, on the contrary, it is the first time that you register, press “**Register now**” .
In the case of a new registration, certain personal data is required.

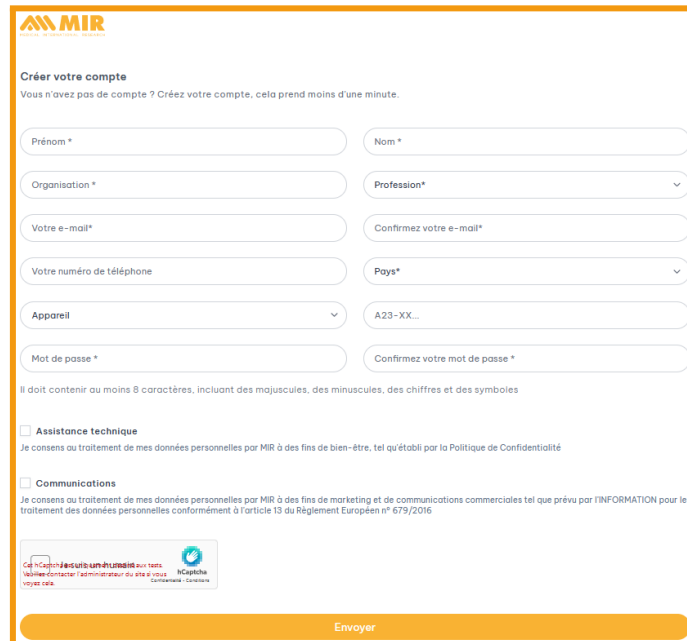

The screenshot shows the MyMir registration page. At the top, there is the MIR logo and the text 'Créer votre compte'. Below this, it says 'Vous n'avez pas de compte ? Créez votre compte, cela prend moins d'une minute.' The form contains several fields: 'Prénom *', 'Nom *', 'Organisation *', 'Profession*', 'Votre e-mail*', 'Confirmez votre e-mail*', 'Votre numéro de téléphone', 'Pays*', 'Appareil' (with a dropdown menu), 'A23-XX...', 'Mot de passe *', and 'Confirmez votre mot de passe *'. Below the form, there are two checkboxes: 'Assistance technique' and 'Communications'. The 'Assistance technique' checkbox is checked, and the 'Communications' checkbox is unchecked. Below the checkboxes, there is a small text block about data processing. At the bottom, there is a 'Envoyer' button.

Accept the agreement for the processing of your personal data by MIR to allow you to benefit from technical assistance

☒ Assistance technique


Je consens au traitement de mes données personnelles par MIR à des fins de bien-être, tel qu'établi par la Politique de Confidentialité

Then press “**Confirm registration**”

An e-mail is sent to the e-mail address you provided. Inside this email is a link that you must click to confirm your email address.

Software Installation

Once logged into your account, you can download the MIR Spiro software, Click on “**Software**” then MIR Spiro and choose “**MIR Spiro pour MacOS**” or “**MIR Spiro for Apple**”. The list of prerequisites is displayed. Then click on “**Download**”

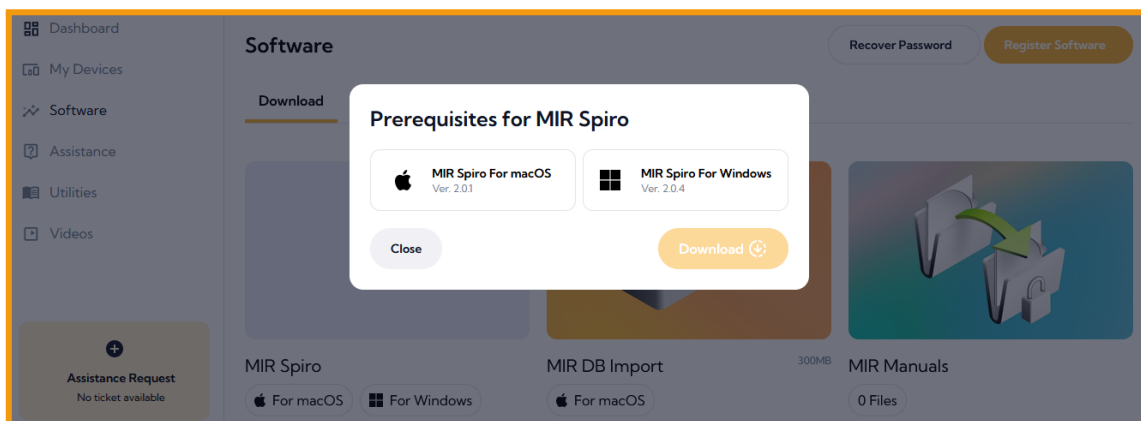


Important: Concerning Apple, you have the possibility to download a tool MirDbImportforApple which will allow you to recover your Winspiro database to MIR Spiro

Download procedure for Windows

Once the MIR Spiro setup.zip file has been downloaded, extract the setup.exe file and launch the setup.exe file **as administrator**

Nom	Taille	Compressé	Type	Modifié	CRC32
..			Dossier de fichiers		
setup.exe	421 066 928	420 532 649	Application	29/04/2022 15:36	742E31C4

To push on “**Next >**” as long as it is possible to do so (Leave the default settings)

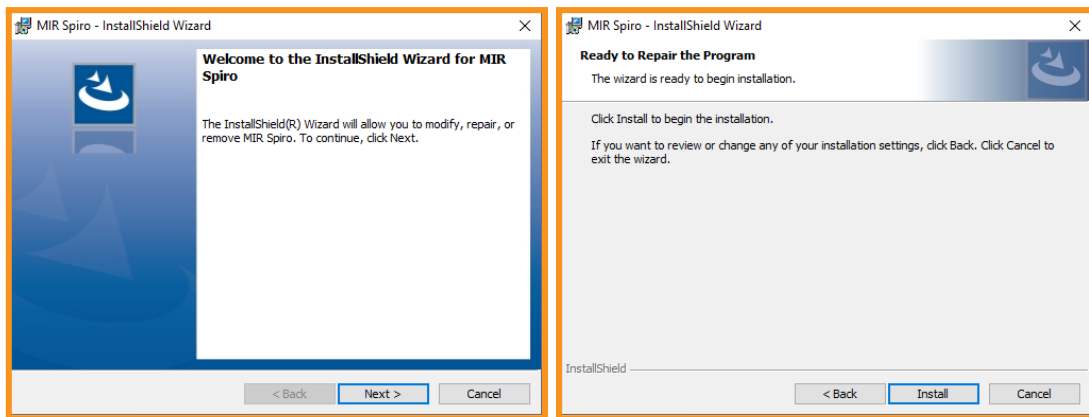


Warning : If you are in an environment prohibiting installations on the C:/ drive (in a hospital for example), you can modify the software installation directory on the 3rd screen.

It is important to know that the software must be physically installed on the client workstation.

Procedure for MAC

Once the MIR Spiro setup_MacOS.zip file is downloaded, extract the setup.pkg and launch the file. Press **Continue**, then **Install**, enter your password (if prompted), and click **Close** to complete the installation.

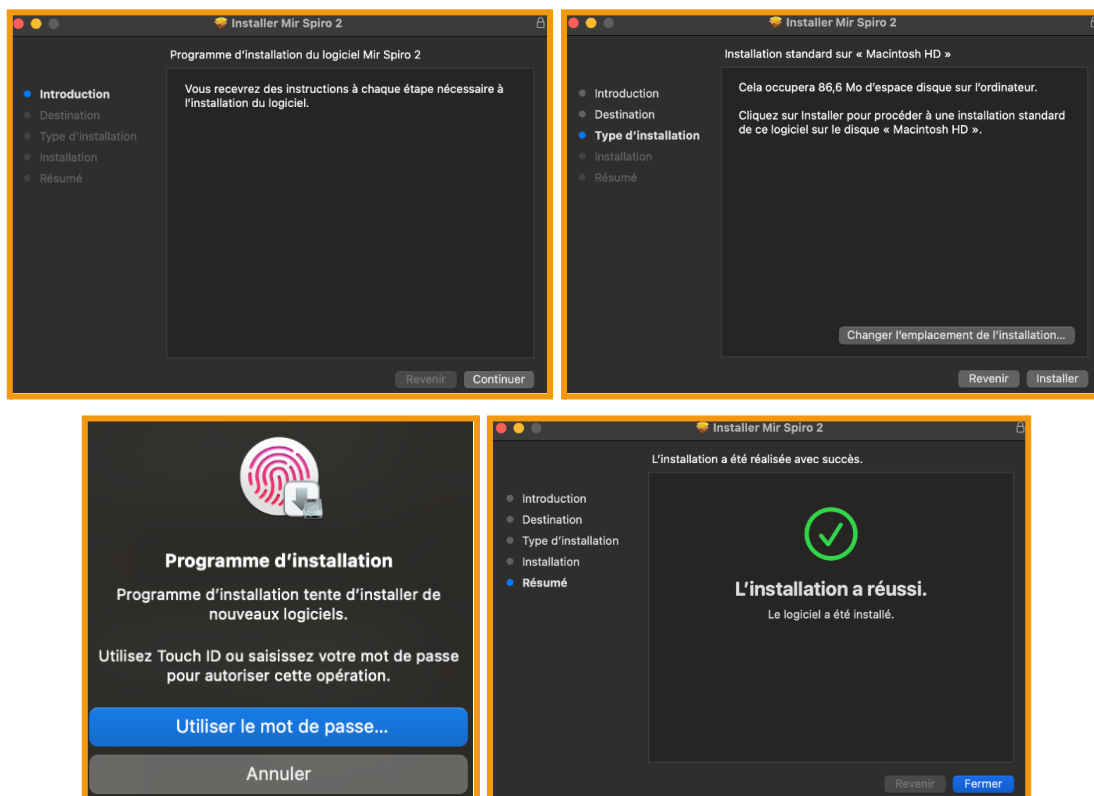

Open the application from the **Launchpad**

Launch MIR Spiro

At the first launch, you must accept the **"Terms & Conditions"** in order to continue. To do this, click on the checkbox then validate by clicking on **"Accept & Continue"** on the next two screens.

TERMS & CONDITIONS

1. DEFINITIONS

"MIR" indicates the company MIR Medical International Research, Via del Magliolino, 125 00155 Roma-Italy.

"Device MIR" indicates the medical device made by MIR.

"Software" means all the content of any files taken electronically by the user from enabled platforms.

"To Use" means to access, install, download, copy or otherwise benefit from the use of the Software functionality.

"Licensee" means a healthcare professional who uses the Software.

2. LICENSE OBJECT

2.1 MIR grants just for health professionals a free use license, not exclusive, and just for software use in correlation with MIR device. If the Licensee agree this EULA on behalf of a company (or other legal entity), they guarantee to have the authority to bind the company. If the Licensee does not have the authority or they do not agree to the EULA terms, neither the Licensee nor the company may use the Software.

☒ Pursuant to articles 1341 and 1342 of the Italian Civil Code, where applicable, the Licensee hereby confirms to have, carefully read and to accept, specifically, the following provisions of the EULA: 3 (Restrictions); 5 (Defects, damages and limitations of liability); 9 (Termination of EULA); 10 (Applicable Law and Jurisdiction).

TERMS & CONDITIONS

For the sole purpose of enabling MIR to improve the Software, including its usability, the Licensee, with his explicit consent, transmits to MIR information relating to:

- Type and version of the operating system used
- IP address
- Anonymous data aggregated for statistical purposes on product use (features used, size of data entered)
- Information on software errors

Information about software errors (errors can also be transmitted by the specific platforms from which the user has obtained the software)

The user, for the same purposes, can send like/dislike on specific product features (the user can choose to hide this function) and respond to surveys on product functionality and propose new ones.

For more information, read the Privacy policy where the Licensee is provided with all information about data processing (https://mymir.spirometry.com/PrivacyPolicyforMIR_SpiroPRO_EN.pdf)

Due to the GDPR law, you have the possibility to define a password to secure access to the MIR Spiro application. **You will be asked for this password each time you open the software.** Click on **"Login with password"** to define your password then validate on **"Register"**

Protection par mot de passe

Voulez-vous protéger votre travail avec la « Protection par mot de passe » ?

Vous pouvez également le faire plus tard

Protection par mot de passe

Nouveau mot de passe

Retapez le nouveau mot de passe

Indice de mot de passe

To activate MIR Spiro, enter the email and password you used when creating your account on **myMIR**. After validation, "Activation successful" window appears on the screen.

Please enter your MyMIR credentials to access the software.

Username

Password

[Password lost ?](#)
[Create an account](#)

MIR Spiro

 Activation réussie

MIR Spiro starts with a window prompting to uninstall a previous version, one asking to import an old database MIRSpiro (version 1.x), and a final one allowing to find a compatible database (type WinspiroPRO.wdb). Select the option of your choice for each window.

MIR Spiro can help you uninstall the previous version.

Do you want to do this now?

Yes
Not now

☐ Don't ask me again

MIR Spiro has detected a 1.x version database that can be imported.

Do you want to do this now?

Yes
Not now

☐ Don't ask me again

MIR Spiro can help you find a compatible database to import

Do you want to do this now?

Yes
Not now

☐ Don't ask me again

Once on the homepage, you can activate your license Platinum. Select **Click here** in the orange bar or click on the menu  then select **Enter code**. Enter your code and **Confirm**. Your license is now activated.

MIR Spiro 2.0

You are using the free version of the software with restrictions and advertisements. Contact your distributor to get your Platinum Subscription Plan or [Click here!](#)

Home Patients Test
Show all patients 

Search patients

Patients

Sort By: Surname (A-Z)

+ New Patient



Search patients

The applied filters did not return any results



Select a patient from the list



Go to Platinum Subscription Plan

Buy here
Enter Code

Enter your Platinum Subscription Code

0 / 8

Don't you have a Platinum Subscription Code yet ? [Buy here](#)

CONFIRM
Close

Congratulations: Your software is now registered and ready to use.

Important: If installing on a new PC, formatting or installing a new operating system, you will need to register again.



ITALIE

MIR Siège social

Via del Maggiolino, 125
00155 Roma

Tél. +39 06 22 754 777
Fax +39 06 22 754 785

mir@spirometry.com

ÉTATS-UNIS

MIR USA, Inc.

5462 S. Westridge Drive
New Berlin, WI 53151

Tél. +1 (262) 565 - 6797
Fax +1 (262) 364 - 2030

mirusa@spirometry.com

FRANCE

MIR Agence locale

Jardin des Entreprises,
290, Chemin de Saint Dionisy
30980 LANGLADE (France)

Tél. +33 (0)4 66 37 20 68
Fax +33 (0)4 84 25 14 32

mirfrance@spirometry.com

www.spirometry.com
www.mirsmartone.com