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Accessories (Optional)			
Nome	Model No	Material	Disinfecting Method: Boiling O: Applicable, X: Not applicable
Nebulizer bottle set (N1 Bottle+Mouthpiece)	NB_AC_001_00	PP	O
Nebulizer bottle (N1)	NB_AC_026_00	PP	O
Mouthpiece	NB_AC_021_00	PP	O
Adult Mask	NB_AC_003_00	PVC	X
Child Mask	NB_AC_004_00	PVC	X
Air Tube (1.5M)	NB_AC_005_00	PVC	X
Air Filters (Ø15x10mm)x5pcs	NB_AC_006_00	PU	X
Nasal Washer	NW1	PP+ Silicone	O
Nasal Aspirator	NS1	PP+ Silicone	O
Filter (nasal aspirator)	NB_AC_032_00	-	X
Air tube (nasal aspirator)	NB_AC_033_00	PVC	X

* Please follow the disinfecting methods to boil and wash the accessories.

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Electromagnetic Compatibility Information

1. This device needs to be installed and put into service in accordance with the information provided in the user manual.
 2. WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the NK1000, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result.
- If higher IMMUNITY TEST LEVELS than those specified in Table 9 are used, the minimum separation distance may be lowered. Lower minimum separation distances shall be calculated using the equation specified in 8.10.

Manufacturer's declaration-electromagnetic immunity			
The NK1000 is intended for use in the electromagnetic environment specified below. The customer or the user of the NK1000 should assure that is used in such and environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
	3 Yrms: 0.15 MHz ~ 80 MHz 6 Yrms in ISM and amateur radio bands between 0.15 MHz and 80 MHz	3 Yrms: 0.15 MHz ~ 80 MHz 6 Yrms in ISM and amateur radio bands between 0.15 MHz and 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the NK1000 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: d = 1.2 √P, d = 1.2 √P 80MHz to 800 MHz, d = 2.3 √P 800MHz to 2.7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	10 V/m 80 MHz ~ 2.7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz ~ 2.7 GHz 80 % AM at 1 kHz	

NOTE1: At 80 MHz and 800 MHz, the higher frequency range applies.
NOTE2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.
More information on EMC compliance of the device can be obtained from Rossmax website: www.rossmax.com.

rossmax

Modèle: NK1000

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www.rossmax.com

Warranty Card

This instrument is covered by a 3 year guarantee from the purchase date. The guarantee is valid only on presentation of the guarantee card completed by the dealer confirming purchase date or the receipt. Nebulizer Accessories are not included. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or noncompliance with the instruction manual. Please contact your local seller/dealer or www.rossmax.com.

Customer Name: _____
Address: _____
Telephone: _____
E-mail address: _____
Product Information
Date of purchase: _____
Store where purchased: _____

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Rossmax Swiss GmbH,